

SURGX™

STERILE ANTIMICROBIAL GEL

DESCRIPTION

SurgX Sterile Antimicrobial Gel is a white, virtually odorless wound gel that helps to maintain a moist environment that is conducive to healing. In laboratory tests, SurgX has been shown to inhibit the growth of microorganisms in the gel.

Use during pregnancy and lactation is not expected to be associated with harmful effects as cutaneous absorption of benzalkonium chloride is minimal, therefore SurgX can be used during pregnancy and in the lactation period under medical supervision.

SurgX contains Benzalkonium Chloride 0.13%, Polyethylene Glycol 400, Polyethylene Glycol 3350, Sodium Citrate, Citric Acid, and Water.

INDICATIONS FOR USE

SurgX is intended for the management of wounds such as post-surgical wounds, stage I-IV pressure ulcers, partial and full thickness wounds, diabetic foot and leg ulcers, first and second-degree burns, grafted and donor sites.

CONTRAINDICATIONS

SurgX should not be used if there is a history of allergy to any of the ingredients.

WARNINGS AND PRECAUTIONS

- For external topical use only.
- Do not cover with alginate dressings.
- Do not ingest.
- Do not use in the eyes or on mucous membranes.
- There is potential for temporary localized burning/stinging at the application site. Contact a physician if the pain persists.
- Benzalkonium chloride may irritate the skin.
- SurgX should not be applied to the skin of the breast during lactation because the baby may take it in with the milk.
- No specific interaction studies have been performed; therefore the use of SurgX with other topical products is not recommended.

INSTRUCTIONS FOR USE

Using sterile technique, SurgX should be applied directly to the wound and then covered with an appropriate dressing. An optional applicator tip for control of gel dispersion is provided in the sterile pouch. The applicator tip is designed to be screwed on to the end of the tube before application.

STORAGE CONDITIONS

Product should be stored between 15°C and 25°C.

www.nextscience.com/patents

Powered by X^{BIO}™ Technology

Manufactured for:

Next Science, LLC













10550 Deerwood Park Blvd

Suite 300

Jacksonville, FL 32256 USA

www.nextscience.com



	Do not use if package is damaged		Do Not Re-Sterilize
	Consult instructions for use		Manufacturer
	Use-by-date		Do Not Re-Use
	Batch code		Temperature storage limit
	Catalogue Number		Caution
	Sterilized using Irridation		Quantity; Box Count
Rx Only	CAUTION: Federal law restricts this product to sale and use by, or on the order of a physician		