

# BLASTX<sup>®</sup>


Antimicrobial Wound Gel



## HOW TO APPLY BLASTX Antimicrobial Wound Gel



**STEP 1**  
**1** **Cleanse**  
Gently cleanse the wound as directed by your healthcare professional

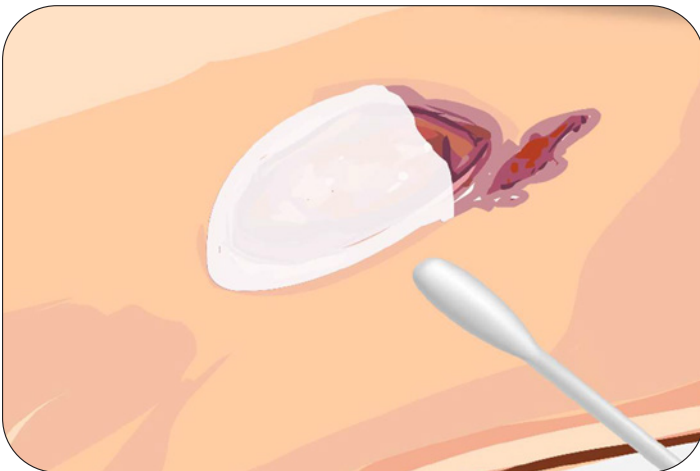

**STEP 2**  
**2** **Apply**  
Apply BLASTX directly to the wound, about 3 mm thick, covering the entire wound area



**STEP 3**  
**3** **Cover**  
Apply a new dressing  
\*do not use alginate dressings

Federal law restricts the device to sale by or on the order of a physician

MPN-WG-008 Rev B

**NEXT SCIENCE<sup>®</sup>**  
#NextScienceHeals