

BLASTX™

ANTIMICROBIAL WOUND GEL

DESCRIPTION

Next Science BlastX Antimicrobial Wound gel is a white and virtually odorless product. BlastX provides wound management by providing a protective barrier that is able to disrupt biofilm structures while maintaining a moist wound environment that is conducive to wound healing. While in place, the antimicrobial properties of the gel inhibit the growth of microorganisms in the product.

BlastX contains benzalkonium chloride 0.13%, polyethylene glycol 400, polyethylene glycol 3350, sodium citrate, citric acid, and water.

INDICATIONS FOR USE

BlastX is intended for the management of wounds such as stage I-IV pressure ulcers, partial and full thickness wounds, diabetic foot and leg ulcers, postsurgical wounds, first and second-degree burns, grafted and donor sites.

CONTRAINDICATIONS

BlastX should not be used if there is hypersensitivity to any of the ingredients.

WARNINGS AND PRECAUTIONS

BlastX can be used during pregnancy and in the lactation period under medical supervision. You should not apply BlastX to skin of the breast during lactation, because the baby may take it in with your milk.

BlastX should not be used if there is a history of allergy to any of the ingredients.

No specific drug interaction studies have been undertaken; therefore use with other topical medications is not recommended.

For external topical use only.

Do not cover with alginate dressings.

Do not ingest.

Must not be used in or near the eyes.

Benzalkonium chloride may irritate the skin.

Potential burning or discomfort may occur.

INSTRUCTIONS OF USE

BlastX should be applied directly to the wound and then covered with an appropriate dressing. Cover entire wound surface with gel at a minimum thickness of 3mm.

After opening, product may be used until labeled expiration date.

STORAGE CONDITIONS

Product should be stored between 15°C and 25°C.

www.nextscience.com/patents

Powered by X^{BIO}™ Technology



Manufactured by:
Next Science, LLC
10550 Deerwood Park Blvd
Suite 300
Jacksonville, FL 32256 USA
www.nextscience.com



Emergo Europe B.V.,
Prinsessegracht 20,
2514 AP The Hague,
The Netherlands



0344

Australian Sponsor

Next Science Pty Ltd
Level 19, Tower A, The Zenith
821 Pacific Highway, Chatsworth
NSW 2067

	Consult instructions for use		Manufacturer
	Authorized Representative in the European Community		Do Not Use if Package is Damaged
	Use-by-date		Temperature Storage Limit
	Batch code		Caution
	Catalogue Number		Medical Device
Rx Only	CAUTION: Federal law restricts this product to sale and use by, or on the order of, a physician.		List of Ingredients