

SURGX[®]

STERILE ANTIMICROBIAL GEL

DESCRIPTION

SURGX Sterile Antimicrobial Gel is a white, virtually odorless wound gel that helps to maintain a moist environment that is conducive to healing. SURGX is designed to reduce surgical site and post-surgical infections by destroying planktonic and biofilm-encased bacteria within the gel.

Use during pregnancy and lactation is not expected to be associated with harmful effects as cutaneous absorption of benzalkonium chloride is minimal. Therefore, SURGX can be used during pregnancy and in the lactation period under medical supervision.

Contains: Benzalkonium Chloride 0.13%, Polyethylene Glycol 400, Polyethylene Glycol 3350, Sodium Citrate, Citric Acid, Water.

INDICATIONS FOR USE

SURGX is intended for the management of wounds such as post-surgical wounds, stage I-IV pressure ulcers, partial and full thickness wounds, diabetic foot and leg ulcers, first and second-degree burns, grafted and donor sites.

WARNINGS AND PRECAUTIONS

- Do not use if there is a history of allergy to any of the ingredients.
- For external topical use only.
- Do not cover with alginate dressings.
- Do not ingest.
- Do not use in the eyes or on mucous membranes.
- There is potential for temporary localized burning/stinging at the application site.
- Benzalkonium chloride may irritate the skin.
- SURGX should not be applied to the skin of the breast during lactation because the baby may ingest the gel.
- No specific interaction studies have been performed; therefore, the use of SURGX with other topical products is not recommended.

INSTRUCTIONS FOR USE

Open pouch using aseptic technique. If applicable, screw on applicator tip to control the gel dispersion. Apply directly to the closed incision site. Ensure complete coverage of the incision. Cover with an appropriate dressing.

STORAGE CONDITIONS

Store between 15°C and 25°C

www.nextscience.com/patents

Powered by **X²BIO** Technology

Manufactured for:

Next Science, LLC













10550 Deerwood Park Blvd

Suite 300

Jacksonville, FL 32256 USA

www.nextscience.com

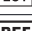

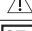	Do not use if package is damaged		Do Not Re-Sterilize
	Consult instructions for use		Manufacturer
	Use-by-date		Do Not Re-Use
	Batch code		Temperature storage limit
	Catalogue Number		Caution
	Sterilized using Irradiation		Quantity; Box Count
Rx Only	CAUTION: Federal law restricts this product to sale and use by, or on the order of a physician		