BLASTX[®]

Antimicrobial Wound Gel

Description

Next Science BLASTX is an antimicrobial wound gel powered by a non-toxic technology that inhibits the growth of microorganisms within the gel application.

BLASTX contains Benzalkonium Chloride 0.13%, Polyethylene Glycol 400, Polyethylene Glycol 3350, Sodium Citrate, Citric Acid, and Water.

Indications for Use

BLASTX is indicated for the management of wounds such as stage I-IV pressure ulcers, partial- and full-thickness wounds, diabetic foot and leg ulcers, post-surgical wounds, first and second degree burns, and grafted and donor sites.

Contraindications

BLASTX should not be used if there is a history of allergy to any of the ingredients.

Warnings and Precautions

For external topical use only. Do not ingest. Must not be used in or near the eyes. Do not cover with alginate dressings.

Instructions of Use

BLASTX should be applied directly to the wound and then covered with an appropriate dressing. BLASTX can be applied once daily for up to 30 days. After opening, product may be used until labeled expiration date.

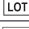
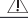
Storage and Handling

Store at 15°C to 25°C (59°F to 77°F). Excursions permitted.

www.nextscience.com/patents

Powered by XBO[®] Technology

Manufactured for:
Next Science, LLC
10550 Deerwood Park Blvd
Suite 300
Jacksonville, FL 32256 USA
www.nextscience.com
Phone: 855-968-6399

	Consult instructions for use		Single patient multiple use
	Use-by-date		Do not use if package is damaged and consult instructions for use
	Batch code		Temperature limit
	Catalogue Number		Caution
Rx Only	CAUTION: Federal (USA) law restricts this product to sale and use by, or on the order of, a physician.		

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