

**NEXT SCIENCE®**



**BLASTX®**  
Antimicrobial Wound Gel

BLASTX Antimicrobial Wound Gel aids in wound management by maintaining a moist wound environment. While in place, the antimicrobial properties of the gel inhibit the growth of microorganisms in the product.

LEARN MORE AT [NEXTSCIENCE.COM](http://NEXTSCIENCE.COM)





## Clinical Benefits

- ✓ Provides a moist wound environment conducive to healing
- ✓ Biocompatible<sup>1</sup>
- ✓ Prevents bacterial growth within the gel<sup>2</sup>
- ✓ Broad-spectrum efficacy<sup>3</sup>



## Indication for BLASTX

BLASTX is indicated for the management of wounds such as stage I-V pressure ulcers, partial- and full-thickness wounds, diabetic foot and leg ulcers, post-surgical wounds, first- and second- degree burns, and grafted and donor sites.



## How to Apply\*


1. Apply a thin layer of BLASTX directly to the wound, covering to the wound edge.
2. Cover with an appropriate dressing.
3. After opening, product may be used until labeled expiration date.



## Case Study\*\*

Patient had a 2-year history of failed wound management with no meaningful improvement. Prior to using BLASTX, standard wound care approaches were unsuccessful. With BLASTX, the wound began progressing toward closure.

- 42-year-old male
- Venous insufficiency & lymphedema on the anterior shin

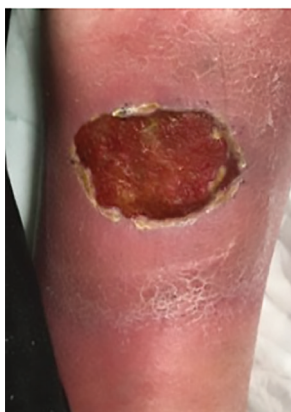

Before use of BLASTX



29 weeks after starting BLASTX



## Case Study\*\*


Patient had history of diabetes mellitus and peripheral artery disease. Before using BLASTX, patient received negative pressure treatment for two months and healing stalled.

- 63-year-old male
- Necrotizing cellulitis and deep abscess



BEFORE

Post Debridement, NPWT Started

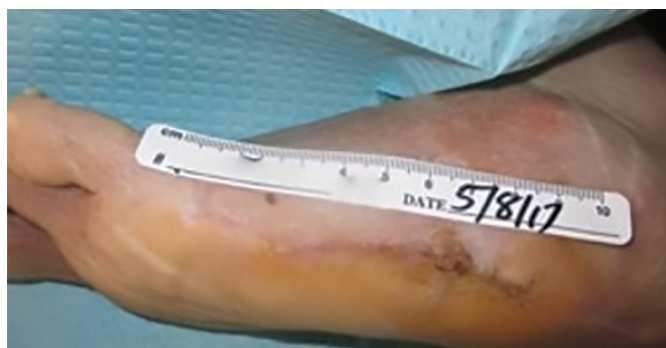

AFTER

Wound closure observed at 16 days

1. Biocompatibility evaluated per ISO 10993-1; Data on file: TR-01-23-003
2. Tested in accordance with USP <51>; Data on file: TR-07-20-003
3. Data on file: TR-03-16-006

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

**CONTAINS** Benzalkonium Chloride 0.13%, Polyethylene Glycol 400, Polyethylene Glycol 3350, Sodium Citrate, Citric Acid, and Water

**CONTRAINDICATIONS** BLASTX Antimicrobial Wound Gel should not be used if there is a history of allergy to any of the ingredients.

**PRECAUTIONS** For external topical use only. Do not ingest. Must not be used in or near the eyes. Do not cover with alginate dressings.

\*Refer to instructions for use for complete information.

\*\*Outcomes can vary by patient and clinical circumstances. Each patient followed course of treatment prescribed by their physician.

**Customer Support:** [cs@nextscience.com](mailto:cs@nextscience.com)

**Orders:** [orders@nextscience.com](mailto:orders@nextscience.com)

**P** (855) 968-6399 **F** (855) 564-2760